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Instructions for Use and Care Knobbed Cushion

Congratulations on purchasing your new TOGU® Knobbed Cushion!

With the Knobbed Cushion, you have chosen a high-quality TOGU® product *made in Germany*. Equipped with stimulating knobs on both sides, the cushion offers a wide range of training and therapeutic possibilities – for both humans and animals.

Product Features

- Shape: round, air-filled cushion with knobs on both sides
Air regulation: adjustable using a standard air pump
Material: durable, high-quality plastic
Manufactured in: Germany

Areas of Application

The TOGU Knobbed Cushion is particularly suited for:

- Finger and hand exercises
Stretching and mobilization
spastic paralysis
Rehabilitation after strokes
Balance training
Coordination training
Obstacle courses to enhance mobility and body awareness
Animal therapy and training
The structured knobbed surface provides a gentle massage effect, enhancing the effectiveness and comfort of each exercise compared to smooth surfaces.

Care Instructions

Clean the Knobbed Cushion using mild, commercially available cleaning or disinfecting agents.

Please do not use:

- Acidic
solvent-based
or corrosive cleaning products, as they may damage the material.

Storage and Temperature Resistance

- Store in a dry, clean location
Avoid direct sunlight and heat sources
Storage temperature: not below -10 °C, not above +45 °C
The material may become softer with heat and firmer with cold

Air Regulation

To adjust the extension resistance, you can inflate or deflate the cushion using a standard air pump. Please moisten the needle slightly before insertion and insert it straight into the valve to avoid damage.